

## N95 MASKS

Worn properly, N95 masks are meant to help block large-particle droplets, splashes, sprays, or splatter that may contain germs (viruses and bacteria) from reaching your mouth and nose.

Respiratory Filters come in a variety of ratings that relate to how the respirator performs (how much it filters) and what environment type it is designed to be used in. When choosing a respirator it's important to understand that as the letter rating and number rating increase, breathing fatigue will increase. Typically, an end user will find an N95 respirator cooler, and easier to breathe through than a P99 or P95 mask.

Respirator Rating Letter Class	Respirator Rating Number Class
<ul style="list-style-type: none"> <li>○ N - Not oil resistant</li> <li>○ R - Resistant to oil</li> <li>○ P - Oil Proof</li> </ul>	<ul style="list-style-type: none"> <li>○ 95 - Removes 95% of all particles that are at least 0.3 microns in diameter</li> <li>○ 99 - Removes 99% of particles that are at least 0.3 microns in diameter</li> <li>○ 100 - Removes 99.97% of all particles that are 0.3 microns in diameter or larger. HE or HEPA quality filter.</li> </ul>

N95 Masks are most commonly used for Flu or Influenza prevention, these masks are effective according to the CDC in helping to prevent the spread of Flu, swine flu and avian flu transmission from person to person. They have added corona virus to the list. N95 masks are designed to fit over the nose and mouth of the wearer, and properly fitted can provide excellent protection.

## Product Package

**For the sake of your health, we suggest that you have a simple 4-step examination before use.**

1. The overall appearance of the mask is damaged or seriously polluted.
2. The headband is not damaged and has good elasticity.
3. There is no damage or break of the nose clip.
4. If there is an exhalation valve, check the condition of the exhalation valve and valve piece to prevent damage or loss.

**Air tightness test, check the tightness between the worn mask and your face**

1. Put your hands together and put your fingers on the mask. Do not move the position of the mask.
2. Breathe significantly, then hold your breath for a few seconds, and you will feel that the mask has a slight inward collapse.
3. Choose hands, hold your smooth for a few seconds, and you can feel the mask bulge outwardly.
4. Only masks that pass the 1-3 step check can meet the air tightness requirements of protective masks.



**WARNING**  
Masks can filter certain contaminants, but they do not eliminate the risk of illness or infection. Improper use of a mask can cause illness or even death. Materials in direct contact with the skin may cause allergic reactions in some sensitive individuals.



**Sunfire ZM950V Earband Particulate Mask**

**Disposable mask is not designed with respiration valve.**

- With adjustable nose clip
- Comfortable to wear
- With improved adaptation on nose
- Ergonomic design
- Closely fitting to the face
- Cotton earband



**N95** Effectively block particles such as dust, bacteria and droplet.

**25** Pieces

**Scope of Use and Storage**

1. Suitable for adults;
2. Not reusable;
3. Store in a well-ventilated, dark, dry environment, keep away from sources of fire and pollution;
4. Storage temperature in the range of -20°C-38°C, the relative humidity is less than 80%;
5. Storage period: 5 years (see the outer box for the production date)

**Wear Instructions**

1. Keep the nose clip facing outwards, and use your both hands to hold the earband on the corresponding side respectively, please make sure that the nose clip is facing upwards;
2. Put on a mask, keep your chin in the mask, and use both hands to hold the earband behind your ears;
3. Adjust to a comfortable position so that the mask can closely fit your face;
4. Press the index and middle fingers of both hands to press and adjust the nose clip until it is close to the bridge of your nose. Perform air tightness test.



## N95 Disposable Protective Mask Without Respiration Valve

### Mask Dimensions

Length: 122  
Width: 112mm  
Height: 2mm  
Weight: 2 grams

### Inner box size

Length: 7"  
Width: 5.5"  
Height: 5"  
Pcs/box: 25

### Master Carton Size

Length: 29"  
Width: 14"  
Height: 11"  
Gross Weight 9 lbs  
Pcs/ctn: 500

# Instruction Sheet



<p><b>Wear Instructions</b></p> <ol style="list-style-type: none"> <li>1. Keep the nose clip facing upwards, and use your both hands to hold the earband on the corresponding side respectively, please make sure that the nose clip is facing upwards.</li> <li>2. Put on a mask, keep your chin in the mask, and use both hands to hold the earband behind your ears.</li> <li>3. Adjust to a comfortable position so that the mask can closely fit your face.</li> <li>4. Press the index and middle fingers of both hands to press and adjust the nose clip until it is close to the bridge of your nose. Perform air tightness test.</li> </ol> <div style="display: flex; justify-content: space-around;">   </div> <div style="display: flex; justify-content: space-around;">   </div> <div style="display: flex; align-items: center; margin-top: 10px;"> <div style="border: 1px solid black; padding: 2px; font-size: 8px;">STERILE</div> <div style="border: 1px solid black; padding: 2px; font-size: 8px;">STERILE</div> <div style="margin-left: 10px;">  <p style="font-size: 8px;">6 973122 980004</p> </div> </div> <div style="margin-top: 10px;">  <p style="font-size: 8px;">Service Hotline: +86-(0)23-52659368 Working hours: 8:00-18:00</p> </div> <div style="margin-top: 10px; border: 1px solid black; padding: 2px; font-size: 8px;"> <p><b>WARNING</b></p> <p>Masks can filter certain contaminants, but they do not eliminate the risk of illness or infection. Incorrect use of a mask can cause illness or even death. Materials in direct contact with the skin may cause allergic reactions in some sensitive individuals.</p> </div>	 <div style="background-color: #008080; color: white; padding: 5px; display: flex; justify-content: space-between; align-items: center;"> <span style="font-size: 24px; font-weight: bold;">Sunfire</span> <span style="font-weight: bold;">ZM950V Earloop Protective Mask</span> </div> <p style="margin-top: 20px;"><b>Disposable mask is not designed with respiration valve.</b></p> <ul style="list-style-type: none"> <li>-With adjustable nose clip</li> <li>-Comfortable to wear</li> <li>-With improved adaptation on nose</li> <li>-Ergonomic design</li> <li>-Closely fitting to the face</li> <li>-Cotton earband</li> </ul> <div style="display: flex; align-items: center; margin-top: 20px;"> <div style="border: 1px solid black; padding: 2px; font-size: 10px; margin-right: 5px;">N95</div> <p style="font-size: 10px;">Effectively block particles such as dust, bacteria and droplet</p> </div> <div style="text-align: right; margin-top: 20px; font-size: 24px; font-weight: bold;">1</div> <p style="font-size: 8px; margin-top: 5px;">Mask Per Pc</p>	<p style="font-size: 8px;"><b>For your health, we recommend that you should perform a simple 3-step check before use.</b></p> <ol style="list-style-type: none"> <li>1. The overall appearance of the mask, e.g. whether it is damaged or heavily polluted;</li> <li>2. The headband is not damaged and has good elasticity;</li> <li>3. No damage or break of the nose clip;</li> </ol> <p style="font-size: 8px;"><b>Air tightness test:</b> As shown in Figure 4, check the tightness between the worn mask and your face.</p> <ol style="list-style-type: none"> <li>1. Put your hands together and put your fingers on the mask. Do not move the position of the mask;</li> <li>2. Inhale vigorously, then hold your breath for a few seconds, and the you will feel that the mask has a significant inward collapse;</li> <li>3. Exhale hard, hold your breath for a few seconds, and the you can feel the mask bulge outwards;</li> <li>4. Only masks that pass the 1-3 step check can meet the air tightness requirements of protective masks.</li> </ol> <div style="border: 1px solid black; padding: 2px; font-size: 8px; margin-top: 10px;"> <p><b>Scope of Use and Storage</b></p> <ol style="list-style-type: none"> <li>1. Suitable for adults;</li> <li>2. Not washable;</li> <li>3. Store in a well-ventilated, dark, dry environment, keep away from sources of fire and pollution;</li> <li>4. Storage temperature: in the range of -20 ℃ - 38 ℃, the relative humidity is less than 80%;</li> <li>5. Storage period: 5 years (see the outer box for the production date)</li> </ol> </div> <div style="text-align: right; margin-top: 10px;">  </div>
---	--	--



The 3rd Party Certificate of  
FDA Medical Device Registration

**Note:**

This file is Not being issued by FDA. We, SFT, as the 3rd party, produce it intended to facilitate customer display & transmit information. The following contents, FDA registered Facility/Owner/Operator&FDA listing Medical Device, are excerpted from database at [www.fda.gov](http://www.fda.gov).

**Establishment:**

1# Plant, [REDACTED]

Registration Number / FEI Number\*:

\* Firm Establishment Identifier (FEI) should be used for identification of entities within the imports message set

Status: **Active**

Date of Registration Status: **2020**

**Owner/Operator**

1# Plant, [REDACTED]

Owner/Operator Number: **10002001**

**Official Correspondent**

Contact Name: [REDACTED]

1# Plant, [REDACTED]

**U.S. Agent**

Contact Name: [REDACTED]

Address: [REDACTED] U.S.A

Phone: [REDACTED] E-mail: [REDACTED].com

**Devices Listing Information**

Proprietary Name	Product Codes	Device Class	Listing Number	Establishment Operations
Mask	LYU	1	D37****	Manufacturer

⚠ Please careful protect your Listing Number.

Approved by: Reilly



Professional FDA Registration Services, by Shanghai Shifu Testing Service Co. Ltd.  
More details on the website: <http://www.sft-lab.com>.  
Need help? Contact us, SFT, at +86(021) 51300821&sales@sft-lab.com.cn  
FDA CERTIFICATE NUM: **SFT20MAR060C**